


Directions for use

PureWick™ Urine Collection System and PureWick™ Female External Catheter


PureWick™ Urine Collection System setup

1. Plug power cord into device outlet and into the A/C power outlet. Note: make sure the power switch is turned off.
2. Place collection canister in the base and press down firmly on the lid of the device.
3. Attach pump tubing (short tube) to the PureWick™ Urine Collection System connector port and the connector port on the collection canister lid.
4. Attach the elbow connector to the collector tubing (long tube) and connect elbow connector to the connector port on collection canister lid.
5. Connect other end of collector tubing (long tube) to a PureWick™ Female External Catheter.

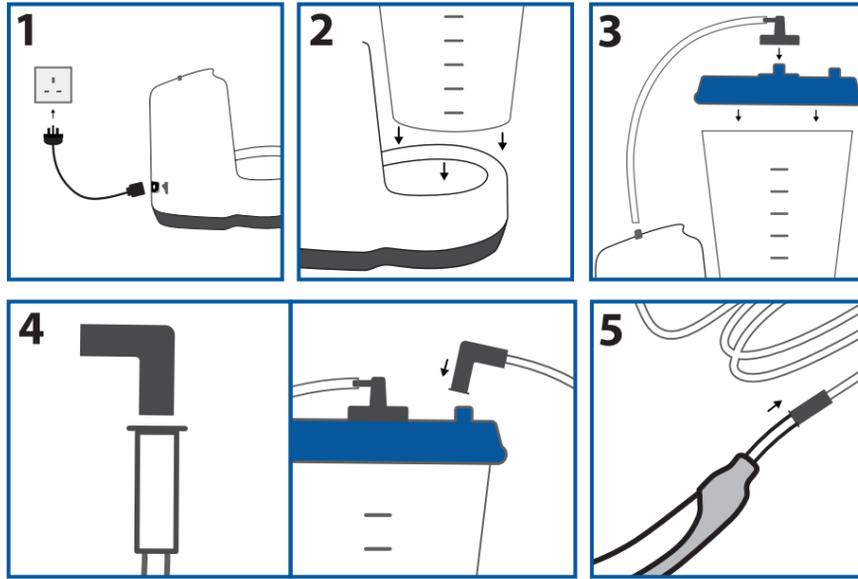


PureWick™ Female External Catheter placement

6. **A.** Perform perineal care and assess skin integrity. **B.** Separate legs, gluteus muscles and labia. Use pubic bone as anatomical marker.
7. **A.** With white soft gauze facing genitalia, align rounded end at gluteal cleft. **B.** Gently tuck soft gauze side between separated gluteus and labia.

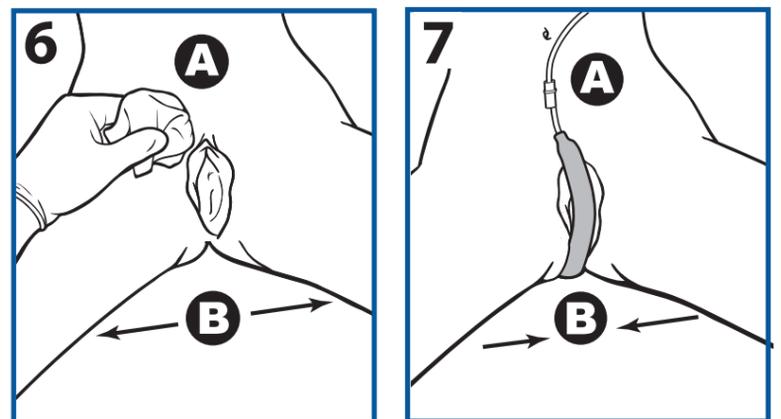

PureWick™ Female External Catheter removal and maintenance

8. Fully separate the legs, gluteus, and labia. **A.** To avoid potential skin injury upon removal, gently pull the PureWick™ Female External Catheter outward. **B.** Make sure all urine has been drawn out of the collector tubing and into the collection canister before removing the PureWick™ Female External Catheter.
9. Replace at least every 8-12 hours or if soiled with faeces or blood. Assess skin for compromise and perform perineal care prior to placement of a new PureWick™ Female External Catheter.

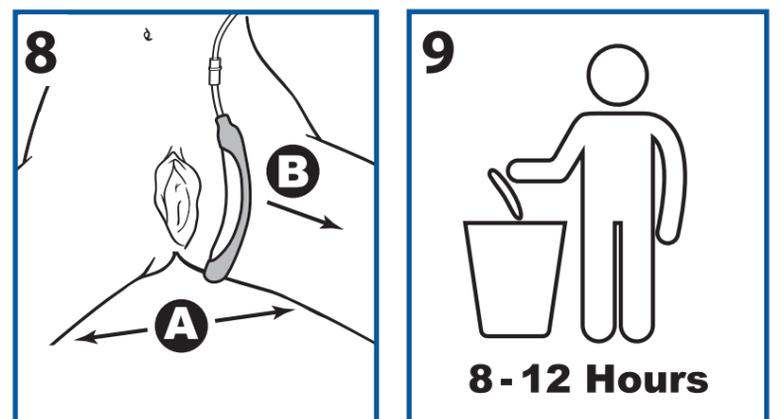

Indications: The PureWick™ Urine Collection System is to be used with the PureWick™ Female External Catheters which are intended for non-invasive urine output management.

Contraindications: The PureWick™ Female External Catheter should not be used on patients with urinary retention.

Warnings: To avoid potential skin injury, never push or pull the PureWick™ Female External Catheter against the skin during placement or removal. Never insert the PureWick™ Female External Catheter into the vagina, anal canal, or other body cavities. Do not use the PureWick™ Female External Catheter with a bedpan or any material that does not allow for sufficient airflow. Discontinue use if an allergic reaction occurs. After use, this product may be a potential biohazard. Dispose of in accordance with applicable local laws and regulations.

Please consult package insert for more detailed safety information and instructions for use.



Freephone:
0800 0121 699

Email:
script-easy@bd.com

Nurse Advice Line Freephone:
0808 168 4048

Web:
www.scripteasy.co.uk

BD, Forest House, Tilgate Forest Business Park,
Brighton Road, Crawley, West Sussex RH11 9BP

www.bardcare.uk

BD, the BD logo, PureWick and Script-easy are trademarks of Becton, Dickinson and Company. Script-easy® is the prescription dispensing service operated by Bard Limited. Bard is now a wholly-owned subsidiary of BD (Becton, Dickinson and Company). © 2020 BD and its subsidiaries. All rights reserved. 0120/3898





PureWick™ System
Directions for use